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Irritant

CEKA BOND is an anaerobic adhesive for the threads of prosthetic precision attachments. Content: 5 ml

CEKA BOND is a quick and efficient adhesive. It is mouth-resistant, non-toxic, and does not dissolve in contact with saliva. Always use **CEKA BOND** to fix threaded connections. This prevents gradual unthreading in the mouth. The individual parts remain interchangeable. Ideal for attachments and implant, prosthetic, and orthodontic components.

Instructions for use

The parts to be connected must be clean and dry. Press a small amount of **CEKA BOND** on a mixing pad. Dip the threads into the material, thread, and remove the excess material from the prosthesis. The prosthesis may be seated immediately. Complete setting is reached after 3 hours. Clean the instruments with water. The threaded connection can be loosened afterwards when necessary. A slight resistance may then be perceptible.

Precautions

Always apply **CEKA BOND** extra orally. Remove any excess material. Rinse with water. Although this product is not known to cause sensitization, contact with skin already damaged must be avoided.

Storage

Limited shelf life: see date on package. Store in original container between 8 °C and 21 °C (46 °F and 70 °F).

Regulatory information

- Irritating to eyes and respiratory system. ·Do not inhale vapours. ·Avoid contact with eyes.
- In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.
- After contact with skin, wash immediately with plenty of water and soap. ·Wear suitable gloves.
- Use only in well-ventilated areas.

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Copyright© ALPHADENT NV 6/2016. Prospectus no 128.N/F/E/D/S/I-ed.F
Copyright© ALPHADENT NV 6/2016. Prospectus no 128.N/F/E/D/S/1-ed.F