APPLICATION

CEKA Multi Composite is a colour-neutral self-curing luting agent for indirect prosthetic assembly. It is an all-round adhesive for multiple dental materials: zirconium oxide, porcelain, composites, titanium, precious and non-precious alloys. Combine always with CEKA Multi Primer. Typical applications are titanium base luting, cylinder luting in implant frames and attachment fixation. **CEKA Multi Primer** contains the newest monomer formula to promote the bonding of dental acrylics and composite resins to implant abutments, zirconium oxide, porcelain, metal frameworks, precision attachments, and orthodontic appliances.

ADVANTAGES

Outstanding bond values to a multitude of dental materials: zirconium oxide, precious and non-precious metals, porcelain, and composites. No light curing necessary. Simple processing by sandblasting first. No etching. Excellent setting behaviour. Easy to apply. Highly flowable material, bubble-free. Automix syringe.

INSTRUCTIONS FOR USE

The two components to be connected are treated in a similar manner. The surfaces have to be cleaned with alcohol or isopropanol. Subsequently, the dried surfaces are sandblasted with aluminium oxide (50 µm, 2 bar) and blown off with oil-free compressed air. Avoid any possibility of contamination (e.g. liquids) and dry. Apply with a dry brush a thin layer of CEKA Multi Primer and let dry for 2 minutes. Use the automix tip to apply the CEKA Multi Composite and spread evenly in a thin layer on the surfaces, then connect the two components. You have maximum 3-4 minutes working time. Remove excess material immediately. Final curing is 30 minutes.

STORAGE

Limited shelf life: see date on label.

Store CEKA Multi Composite refrigerated between 3 °C and 9 °C (37 °F and 48 °F).

Store CEKA Multi Primer in a cool place between 3 °C and 25°C (37 °F and 77 °F).

CONTENTS / REORDER INFORMATION

6700 CEKA Multi Set: CEKA Multi Composite 2 x 4 g, 6 automix tips, CEKA Multi Primer 5 ml, brush

6701 CEKA Multi Composite 2 x 4 g, 6 automix tips

6702 CEKA Multi Primer 5 ml

6705 CEKA Multi Automix tips 20 pcs

Find a distributor nearby on www.CKPL.eu.

PRECAUTIONS / REGULATORY INFORMATION FOR CEKA MULTI PRIMER

Highly flammable • Irritating to respiratory system and skin • May cause sensitization by skin contact • Keep container in a well-ventilated place • Keep away from sources of ignition • No smoking • Do not breathe fumes • Avoid contact with skin • Wear suitable gloves • This material and its container must be disposed of as hazardous waste

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