**APPLICATION**

**CEKA Multi O Composite** is an opaque coloured self-curing luting agent for indirect prosthetic assembly. It is an all-round adhesive for multiple dental materials: zirconium oxide, porcelain, composites, titanium, precious and non-precious alloys. Combine always with CEKA Multi Primer. Typical applications are titanium base luting, cylinder luting in implant frames and attachment fixation. Ideal for cementing translucent materials (e.g. e-max) as the opaque layer prevents the (titanium) abutment from shining through the crown. **CEKA Multi Primer** contains the newest monomer formula to promote the bonding of dental acrylics and composite resins to implant abutments, zirconium oxide, porcelain, metal frameworks, precision attachments, and orthodontic appliances.

**ADVANTAGES**

Outstanding bond values to a multitude of dental materials: zirconium oxide, precious and non-precious metals, porcelain, and composites. No light curing necessary. Simple processing by sandblasting first. No etching. Excellent setting behaviour. Easy to apply. Highly flowable material, bubble-free. Automix syringe.

**INSTRUCTIONS FOR USE**

The two components to be connected are treated in a similar manner. The surfaces have to be cleaned with alcohol or isopropanol. Subsequently, the dried surfaces are sandblasted with aluminium oxide (50 µm, 2 bar) and blown off with oil-free compressed air. Avoid any possibility of contamination (e.g. liquids) and dry. Apply with a dry brush a thin layer of CEKA Multi Primer and let dry for 2 minutes. Use the automix tip to apply the CEKA Multi O Composite and spread evenly in a thin layer on the surfaces, then connect the two components. You have maximum 3-4 minutes working time. Remove excess material immediately. Final curing is 30 minutes.

**STORAGE**

Limited shelf life: see date on label.

Store CEKA Multi O Composite refrigerated between 3 °C and 9 °C (37 °F and 48 °F).

Store CEKA Multi Primer in a cool place between 3 °C and 25 °C (37 °F and 77 °F).

**CONTENTS / REORDER INFORMATION**

# 6710 CEKA Multi O Set: CEKA Multi O Composite 2 x 4 g, 6 automix tips, CEKA Multi Primer 5 ml, brush

# 6711 CEKA Multi O Composite 2 x 4 g, 6 automix tips

# 6702 CEKA Multi Primer 5 ml

# 6705 CEKA Multi Automix tips 20 pcs

Find a distributor nearby on [www.CKPL.eu](http://www.CKPL.eu).

**PRECAUTIONS / REGULATORY INFORMATION FOR CEKA MULTI PRIMER**

Highly flammable • Irritating to respiratory system and skin • May cause sensitization by skin contact • Keep container in a well-ventilated place • Keep away from sources of ignition • No smoking • Do not breathe fumes • Avoid contact with skin • Wear suitable gloves • This material and its container must be disposed of as hazardous waste

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