CEKA AXIAL
 PRECI-CLIX AXIAL
 PRECI-CLIX RADICULAR
 PRECI-BALL

01.
 AXIAL
 ATTACHMENTS

CEKA EXTRACORONAL
 PRECI-VERTIX
 PRECI-CLIX EC
 PRECI-SAGIX
 PRECI-52


02.
 EXTRACORONAL
 ATTACHMENTS

PRECI-BAR
 PRECI-HORIX
 PRECI-CLIP

03.
 PROFILE
 ATTACHMENTS

PRECI-PROFILE
 PRECI-POST
 CEKA SOL
 CEKA SITE
 CEKA BOND
 3C-BOND
 PERMA-RET
PRECI-SEP
 PLASTICWAX
 EXPANDO
 CEKA Multi (O)
 MEASURING GAUGE
 CKPL DIGITAL LIBRARY

SPECIALTIES

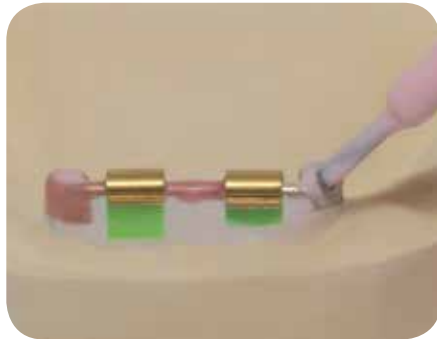WWW.CKPL.EU

PRECI-SEP
 EN-DE-FR-NL-IT-ES

SIDE EFFECTS, WARNINGS AND PRECAUTIONS

The attachments are intended for single use.
 The products are non-sterile.
 There is a risk of poor fit when patient conditions change.
 Bacterial adhesion can be avoided by applying hygiene measures.
 Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
 The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
 For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
 Do not heat products containing titanium.
 Do not use products containing nickel in case of nickel allergy.
 The accessories RE H 79 and H 35 must be used outside the mouth.

PRECI-SEP

PRECI-SEP is a latex-based separator, spacer and release agent for multiple dental applications:

- use as a spacer for precision attachments
- use as a spacer for composite and acrylic provisional restorations
- use as a spacer for customized abutments when modelling with self-cure or light-cure acrylic
- protects against sandblasting and acids
- use as a spacer with vacuum formed trays
- release agent for crowns when pouring a pick-up impression

Apply with a brush and let dry. To speed up drying, use compressed air. The pink colour will become translucent. Use multiple layers when more thickness is desired. Peels off clean.

Use a few drops of distilled water if PRECI-SEP thickens.

CAUTION: contains natural rubber and may cause allergic reactions.

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