

PROCESSING PRECI-BALL AVK

1. Wax up the post-copings. Solder the **1203 C** males to the post-copings (see INFO 048, items 1-8).
2. Screw the **1207 A** female into the **694 AKS2** retention part (always using the **IMP-XS-042** screw-driver) and place the female on the male.
3. Position the **RA 0055** large space maintainer.
4. Polymerize the female into the prosthesis.
5. Adjust the retention after finishing the prosthesis (see INFO 053).

Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.