

## PROCESSING THE PRECI-HORIX HOUSING

### INCORPORATION OF A HOUSING INTO A NEW PROSTHESIS

1. Make the abutment retainers with PRECI-HORIX (see INFO 002).
2. Place the housings on the space maintainers before closing the flask and seating the prosthesis. They must be fully seated. Polymerize the space maintainers.
3. Remove the space maintainers after polymerization with a scalpel.
4. Seat the riders with the **1705** insertion tool in position.
5. See INFO 016 to select the rider with the proper retention.

### INCORPORATION OF A HOUSING INTO AN EXISTING PROSTHESIS

1. Take impression with the riders seated in the prosthesis.
2. Reduce the **1707** pattern analogue to its correct length to allow for positioning into the impression. Provide the underside which is positioned in the plaster with retention (using a carbide or diamond burr).
3. Pour a model in stone and remove the prosthesis. Remove the impression material and the riders from the prosthesis.
4. Cut a lingual or palatal opening in the prosthesis in the area of the riders.
5. Place the **1703** space maintainers on the model. (They must accurately fit the plastic pattern.) Block out the undercuts and place the housings on the space maintainers.
6. Lubricate the model and reposition the prosthesis on the plaster model.
7. Cure with self-curing resin and polymerize.
8. Remove the prosthesis after hardening of the acrylic resin and finish.
9. Press in the required riders with the **1705** insertion tool.
10. See INFO 016 to select the rider with the proper retention.

### Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.