

REPLACING A PRECI-VERTIX FEMALE

The **1801** and **1811** PRECI-VERTIX attachments are supplied with the **1802** yellow standard females.

If the friction of the female no longer meets the requirements, it can be replaced with the **1805** white female with reduced friction or the **1806** red female with increased friction.

1. Use a small sharp instrument to remove the female from the prosthesis.
2. Position the required female on the supplied **1804** insertion tool and press it into the prosthesis.
3. Check if the required friction is attained.

Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.