

## **REBASING/RELINING A PROSTHESIS WITH PRECI-VERTIX**

1. Place impression material on the gingival of the prosthesis and take a reline impression.
2. Remove the females with a sharp metal instrument from the prosthesis.
3. Completely fill up the metal sleeves with silicone.
4. Pour a stone model.
5. Reline the prosthesis in the usual manner. Use a reline flask or jig.
6. Remove the silicone from the metal sleeves and seat the new females (see INFO 026).

### **Side effects, warnings and precautions**

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.