

FABRICATION OF A NEW PROSTHESIS WITH PRECI-VERTIX

1. Take a full arch impression.
2. Seat the **1819** pattern analogue in the cavity. The round opening is used for retention in the stone.
3. Seat the replaceable **1802** female over the male. Block out the undercuts of the attachment slightly conical with wax.
4. Proceed as described in INFO 014 or INFO 015.

Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.