

REBASING/RELINING A PROSTHESIS WITH PRECI-CLIP

1. Take a reline impression with the riders in the prosthesis.
2. Fill up the area of the riders and the bar with silicone, and pour a stone model.
3. Reline as usual. Use a reline flask or jig.
4. Activate or deactivate the riders after finishing and polishing if necessary (see INFO 107).

Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.