

ACTIVATING OR DEACTIVATING A FEMALE IN A PROSTHESIS WITH PRECI-BALL

ACTIVATING/SQUEEZING THE FEMALE SEGMENTS

Press the **1220** activating tool carefully and step by step over the female, until the desired increased retention is attained.

DEACTIVATING/SPREADING THE FEMALE SEGMENTS

Press the **1221** deactivating tool carefully and step by step over the female, until the desired decreased retention is attained.

Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.