

WORKING DUMMIES/ANALOGUES

To prevent spring pins from being damaged during processing, working dummies with reduced friction or retention levels are available in the CEKA Attachment System. These dummies replace the original spring pin during fabrication of the removable partial or full denture.

Two different working dummies are available:

- **RE H 1:** for **REVAX M 2** spring pins **RE 0031**
Use the **RE H 5** laboratory key.
- **H 1:** for **REVAX M 3** spring pins **694 C**
Use the **RE H 5** laboratory key.

The spring pins are always supplied with a space maintainer.

Always replace the working dummy at the end of the working procedure with the original spring pin (see INFO 063). Do not forget to secure the spring pin with **CEKA BOND** (see INFO 069).

Attention: the working dummies are made of non-precious alloys.
They must **never** be used in the **patient's mouth**.

Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.