

REPLACING A PRECI-CLIX FEMALE

PRECI-CLIX attachments are supplied with the **1231** yellow standard females.

When the retention of the female no longer meets the requirements, it can be replaced with the **1232** white female with reduced retention or the **1233** red female with increased retention.

1. Bend the lips of the female with a sharp instrument towards the inside.
2. Carefully drill with the **IMP-XH-135** head-cutting burr a hole in the centre of the female (up to the housing).
3. Remove the female with a sharp instrument.
4. Position the desired female on the **1222** insertion tool and press this assembly into the housing.
5. Check if the required retention is attained.

Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.