

## REPLACING A PRECI-SAGIX FEMALE

All PRECI-SAGIX introduction kits are supplied with females with 3 different levels of retention.

When the retention of the female does not meet the requirements, it can be replaced with a **1312** (Mini  $\varnothing$  1.7 mm) or **1362** (Standard  $\varnothing$  2.2 mm) white female with reduced retention, a **1311** (Mini  $\varnothing$  1.7 mm) or **1361** (Standard  $\varnothing$  2.2 mm) yellow female with normal retention, or a **1313** (Mini  $\varnothing$  1.7 mm) or **1363** (Standard  $\varnothing$  2.2 mm) red female with increased retention.

1. Use a pointed instrument to remove the female.
2. Steam clean the cavity in the frame.
3. Insert the female with the desired level of retention using the **1323** universal insertion tool (for  $\varnothing$  1.7 mm and 2.2 mm).
4. Check if the female is properly seated and if the desired retention is obtained.

### Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.