

REPLACING A PRECI-52 MALE PART

1. Unthread the **1563** (U - Mini \varnothing 3.15 mm) or **1568** (S - Standard \varnothing 3.40 mm) internal male set using the **1570** star-shaped screw-driver.
2. Thread a new **1563** (U - Mini \varnothing 3.15 mm) or **1568** (S - Standard \varnothing 3.40 mm) internal male set into the male housing using the **1570** star-shaped screw-driver.

Side effects, warnings and precautions

- The attachments are intended for single use.
- The products are non-sterile.
- There is a risk of poor fit when patient conditions change.
- Bacterial adhesion can be avoided by applying hygiene measures.
- Inappropriate use or bad manufacturing can lead to premature wear of the attachments.
- The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.
- For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.
- Do not heat items containing titanium.
- Do not use items containing nickel in case of nickel allergy.
- The accessories RE H 79 and H 35 must be used outside the mouth.