

CEKA AXIAL  
 PRECI-CLIX AXIAL  
 PRECI-CLIX RADICULAR  
 PRECI-BALL

**01.**  
 AXIAL  
 ATTACHMENTS

CEKA EXTRACORONAL  
 PRECI-VERTIX  
 PRECI-CLIX EC  
 PRECI-SAGIX  
 PRECI-52


**02.**  
 EXTRACORONAL  
 ATTACHMENTS

PRECI-BAR  
 PRECI-HORIX  
 PRECI-CLIP

**03.**  
 PROFILE  
 ATTACHMENTS

PRECI-PROFILE  
 PRECI-POST  
 CEKA SOL  
 CEKA SITE  
 CEKA BOND  
 3C-BOND  
 PERMA-RET  
**PRECI-SEP**  
 PLASTICWAX  
 EXPANDO  
 CEKA Multi (O)  
 MEASURING GAUGE  
 CKPL DIGITAL LIBRARY

**SPECIALTIES**

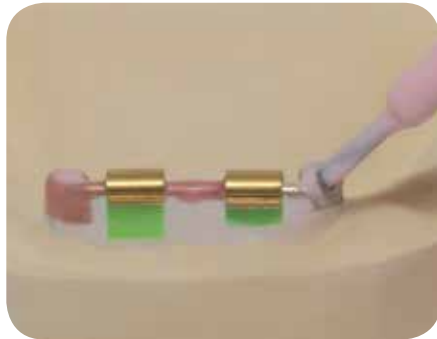

[WWW.CKPL.EU](http://WWW.CKPL.EU)

**PRECI-SEP**  
 EN-DE-FR-NL-IT-ES

**SIDE EFFECTS, WARNINGS AND PRECAUTIONS**

The attachments are intended for single use.  
 The products are non-sterile.  
 There is a risk of poor fit when patient conditions change.  
 Bacterial adhesion can be avoided by applying hygiene measures.  
 Inappropriate use or bad manufacturing can lead to premature wear of the attachments.  
 The functionality of the attachments will be adversely affected by traumas such as grinding and bruxism.  
 For the purpose of traceability we advise you to record the lot number of the applied products in the patient file.  
 Do not heat products containing titanium.  
 Do not use products containing nickel in case of nickel allergy.  
 The accessories RE H 79 and H 35 must be used outside the mouth.



# PRECI-SEP



**PRECI-SEP** is a latex-based separator, spacer and release agent for multiple dental applications:

- use as a spacer for precision attachments
- use as a spacer for composite and acrylic provisional restorations
- use as a spacer for customized abutments when modelling with self-cure or light-cure acrylic
- protects against sandblasting and acids
- use as a spacer with vacuum formed trays
- release agent for crowns when pouring a pick-up impression

Apply with a brush and let dry. To speed up drying, use compressed air. The pink colour will become translucent. Use multiple layers when more thickness is desired. Peels off clean.

Use a few drops of distilled water if PRECI-SEP thickens.

**CAUTION:** contains natural rubber and may cause allergic reactions.

For lab use only.

**# 6501 15 ml**

# PRECI-SEP



# PRECI-SEP

